

AEG ID

Manual

ARE H3.0



1.	Introduction	3
2.	ARE H3.0	3
2.1	Use Case	3
2.2	Functionality	4
2.2.1	General usage	4
2.2.2	Power supply	5
2.2.3	Interface	5
2.2.4	Options	6
2.2.5	Setup tool	7
3.	FCC Statement	13
3.1	ARE H3.0	13
4.	CE statement	13
5.	Release, Change Protocol	13

1. Introduction

ARE H3.0 is a compact handheld ISO 11784/85 reader. It reads ISO 11784/85 compatible transponders as well as ASK 64 bit transponders. In addition, the reader displays temperature if a temperature sensor is present in the transponder. Optionally programmed data are also displayed.


2. ARE H3.0

2.1 Use Case

ARE H3.0 is used to read ISO transponders in animals. Typically, this is done in a veterinary environment or in animal shelters.

ARE H3.0 is not meant for any other use case.

Typical use case is identifying companion animals in a vet or shelter environment.

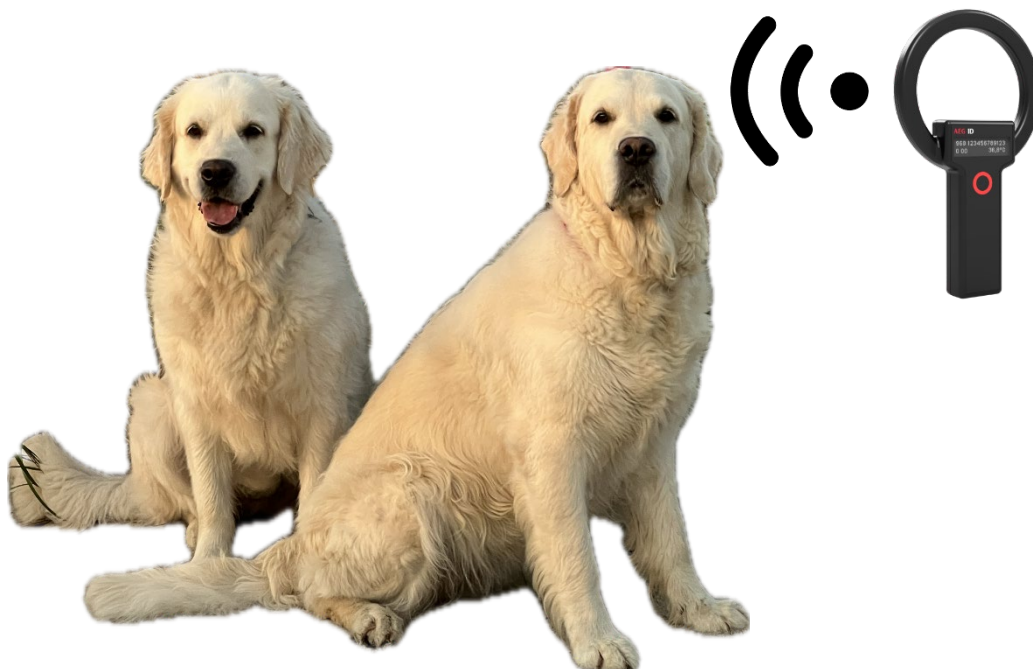

2.2 Functionality

2.2.1 General usage

Reading a transponder

Pressing the red button switches on the reader. The reader may display the last number read before switch off for convenience reasons. If no number was read before switch off, it displays name and firmware version.

Each subsequent press of the button triggers a read. The reader searches for a transponder for a preset time. Once the transponder is read, the ISO number together with retagging counter and user information field and temperature if present, is displayed.

If the reader does not detect a transponder, the message “no tag” is displayed.

Antenna:

Hold ring shaped antenna towards injection site of transponder in animal

For companion animals this is the left side of the head towards the shoulder (see ISO15639 for details)



Display:

2x16 characters LCD screen
Displays ISO number, optional data and temperature

Red-Black button:

Press to switch on and press again to read

2.2.2 Power supply

ARE H3.0 runs off an internal, rechargeable Li- Po battery. This battery cannot be exchanged. Opening the housing voids any warrantee or guarantee.

Use a standard USB-C to USB-C or a USB-C to USB-A cable and a standard 5V USB power supply to recharge the internal battery. Any smartphone power supply will work here fine. Do not use any other means of charging. The device may be damaged. ARE H3.0 indicates the status of the battery at each startup. Whenever a USB-C cable is plugged in, the reader displays its charging status also.


Start display

ARE H3.0 fdx-b	
v1.1.7.6	100%

Charging display

Charging. . .	
	80%

2.2.3 Interface

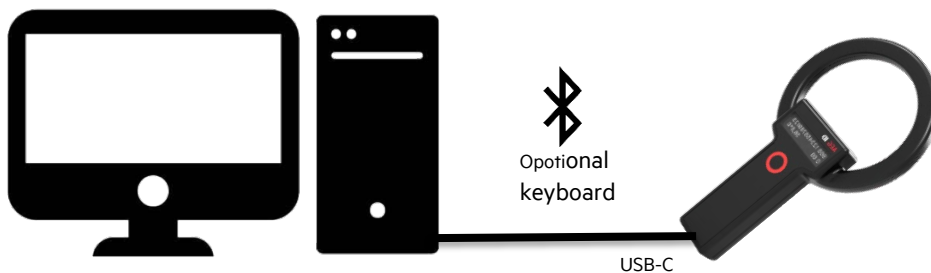

A. USB-C

ARE H3.0 offers a USB-C type interface for communication and charging. When plugged in, the reader installs itself as serial interface or as HID keyboard depending on its setting. Pressing the button for more than 5 seconds switches the interface from serial to keyboard and back.

Pressing button for ~5 seconds

Output:
HID keyboard

Pressing button again for ~5 seconds

Output:
serial

Serial interface:

ARE H3.0 sends the read data out of its serial interface. The format can be influenced. Please see section 2.2.5 for details. AEG ID PC program "ARE H3.0 setup tool" can be used to communicate to and setup ARE H3.0 reader. Please see section 2.2.5 for details.

USB-HID keyboard:

ARE H3.0 can operate as a regular keyboard. ARE H3.0 sends the read data out to the input focus of the PC operating system, just as a regular keyboard would. The format can be influenced. Please see section 2.2.5 for details.

B. Bluetooth:

Some models of ARE H3.0 offer a Bluetooth HID keyboard functionality. If present, search for Bluetooth device AEGIDxxxxxx (xxxxxx represent the serial number of the reader). Once connected, the reader sends read transponder data via Bluetooth HID keyboard to the connected device. The format can be influenced. Please see section 2.2.5 for details. In Bluetooth HID mode the reader vibrates first for each read and a second time to signal successful transfer of data via Bluetooth. Please note, this may take a few seconds depending on system Setup. Standard PIN is 0000.

2.2.4 Options

ARE H3.0 is a LF 134.2kHz reader. The following chip technologies can be read:

Transponder types

ISO 11784/85 fdx-b

ISO 11784/85 hdx (model depending)

ISO BDE (waste management standard)

ASK 64 Bit read only

Some ISO 11784/85 transponders support temperature measurement. ARE H3.0 supports those transponders.

Temperature transponder types

Type 1 (based on EM 4115)

Type 2 (based on Excelio EL9265 – model depending)

Temperature unit

Temperature can be displayed in Celsius or Fahrenheit.

Celsius

968 123456789123
0 00 38.0

Fahrenheit

968 123456789123
0 00 100.4

Data function

Some AEG ID ISO 11784/85 transponders support 16 characters of ASCII data to be programmed into the chip. This data function can be used to write the telephone number of the pet owner into the chip. Alternatively the name of the animal or any other relevant data (e.g. birth date) for that animal can be written into the chip. ARE H3.0 displays these 16 characters in the second line of the display.

Examples:

968 123456789123
0731-140088-0AEG

968 123456789123
ELSA 24.01.2021

Please note, if the transponder also measures temperature, the temperature is displayed in the second line of the display in exchange with data

Display data for 3 seconds

968 123456789123
0731-140088-OAEG

Display temperature for 1 second

968 123456789123
38.0

Serial output string

Serial output string can be set acc. IS 24631-6 or can be customized. Please see section 2.2.5 for details.

Display of last number read after power up

The reader displays the last number successfully read before shut off after being switched on again. This is helpful in a vet situation, where the animal only needs to be read once without regard about the switch off time of the reader. In case there was no number read before shut off, the reader displays its name and software version.

2.2.5 Setup tool


AEG ID PC program “AEG ID ARE H3.0 terminal” can be used to determine optional settings for the reader. Start the program by clicking on its icon. The program opens up.



Interface:

Open the correct interface. Check in system settings which is the correct port.


Serial parameters are 19.200 baud, 8 bits, 1 stop bit, no parity, no hardware control. Use config button to open serial settings.

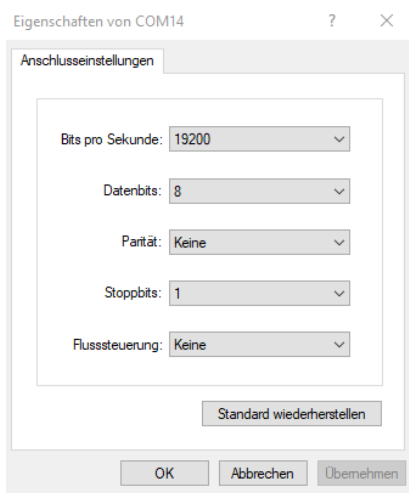

Terminal function:

Once the correct COM port is successfully opened, type ver in the edit field and press “SendCmd”.

The reader answers with its name and serial number.

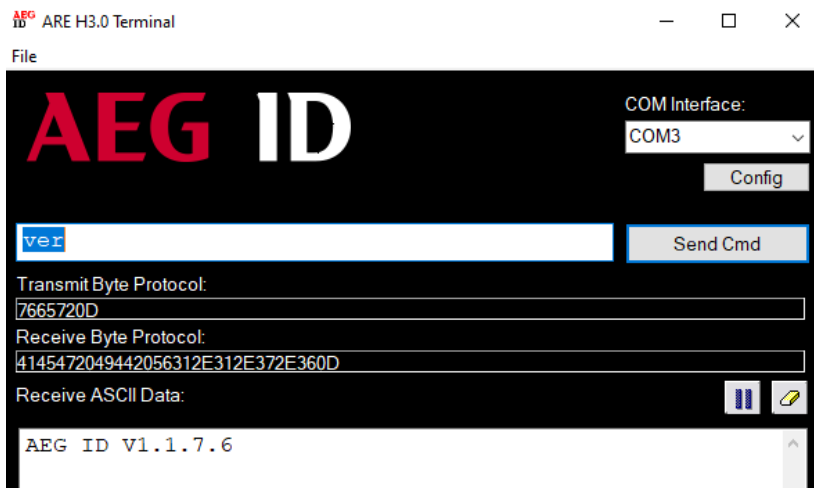

When you see a similar output, communication with ARE H3.0 is established.

The terminal function can be used to communicate directly with ARE H3.0. Instructions are typed in the edit field and then are sent by pressing the button “SendCmd”.

It is recommended to use the setup function of this program to configure the reader (see next section).

Data settings

Checkbox “enable data” switches on data display capabilities of ARE H3.0 to display data in the second line of the display. Only valid data with correct checksum are displayed.

Type in any combination of up to 16 alphanumeric characters in the edit field. If you enter less than 16 characters, the program automatically adds 0x00 for each missing character starting from left to right.

Put the transponder in the reading field of ARE H3.0 (around 5cm distance perpendicular in the middle of the round antenna).

Press button “program data on tag” and wait for the reader to acknowledge the programming by coloring the edit field green. If anything went wrong, the edit field turns red.

Transponder type settings

You can set any combination of the available transponder types to be read by checking the corresponding check boxes.

Temperature settings

Select the temperature transponder you want to read (either none, type 1 or type 2). Only one can be selected at any time. Select the temperature unit you want to be displayed. Only one can be selected at any time.

None:

No temperature information will be displayed or sent.

Temperature Type 1:

Select this setting for EM 4115 based transponders. Here the temperature information is automatically put into the sector trailer of the ISO 11784/85 message when reading a transponder. Read range is comparable to regular ISO transponders.


Temperature Type 2:

Select this setting for EL9265 based transponders. Here the transponder requires an additional get temperature instruction to send its temperature. This reduces the read range considerably compared to regular ISO transponders. Reading time is also increased. This is due to the nature of the chip and can’t be prevented. EL09265 has different configuration options to retrieve temperature. This device supports the most common setup. **ATTENTION: In this setting only EL9265 transponders can be read, regular ISO transponders require Temperature Type setting of none.**

Check dbb:

Some temperature transponders on the market are not programmed correctly (data block bit (dbb) is not set, despite temperature data being added into the sector trailer. ARE H3.0 normally checks for data block bit to be set with temperature transponders. If it is not set, temperature is not displayed. In order to be able to read wrongly programmed transponders nonetheless, this check can be disabled.


Keyboard settings

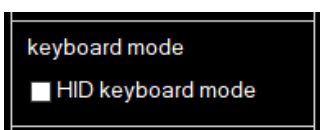

Check USB- HID if you want to use the wired keyboard functionality. Please note, switching back from HID keyboard to serial mode can only be done by pressing the button for more than 5 seconds.

If your reader has Bluetooth option installed, check Bluetooth-HID to enable Bluetooth keyboard.

Switch off time setting

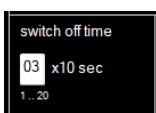

Here you can determine the switch off time from 10 seconds (1) to 200 seconds (20). The edit field only accepts values from 1 to 20.

Reading Time settings



Here you can determine the reading time from 1 second to 25 seconds. The reader searches for transponders automatically during this reading time.

Vibration signal setting



Some animals are scared by sound, so vibrating signal can be disabled.

separator 2

You can determine up to 2 ASCII characters as separator between the ISO number and the temperature if applicable. Default is one comma (2C00). If you enter only one character, the other character will be set to 0x00 from left to right.

post characters

You can determine up to 2 post characters after the ISO number and temperature. Default is carriage return, line feed (0x0D0A). If you enter only one character, the other character will be set to 0x00 from left to right.

The settings are valid for both serial and keyboard interface.

Serial output with above parameters:

968 123456789012,38.2<CR><LF>


Actual string on interface:

393638203132333435363738393031322C33382E320D0A

Data transmission via keyboard functionality (USB and Bluetooth)

Above settings are also valid for keyboard functionality. Please note, not all serial ASCII control characters are available as keystrokes. Here the equivalent will be transmitted.

Setup of ARE H3.0

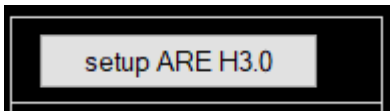

Press button "setup ARE H3.0" to transmit all of above settings to the reader. Please note, all settings will be automatically saved in EEPROM memory of ARE H3.0. A Message box appears to confirm successful setup.

After setup, ARE H3.0 can be used as intended.

3. FCC Statement

3.1 ARE H3.0

Valid for ARE H3.0

Federal Communications Commissions (FCC) Statement

§15.21

You are cautioned that changes or modifications not expressly approved by the part responsible for compliance could void the user's authority to operate the equipment.

§15.105 Information to the user.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

4. CE statement

ARE H3.0 is meant to be used in a veterinary environment for occasional reading of ISO transponders embedded in animals. It is expressly not meant to read transponders embedded in or on humans. This use case is not allowed.

5. Release, Change Protocol

Revision:	Date:	Changes:	Author:
01	08.10.2024	Release first edition	NK
02	23.10.2024	Details added	NK
03	04.12.2024	Details added	NK
04	03.03.2025	Details added	NK
05	21.05.2025	Details added	NK



AEG Identifikationssysteme GmbH
Hörvelsinger Weg 47
89081 Ulm

Tel.: +49 731 14 00 88 – 0

Email: sales@aegid.de

Web: www.aegid.de

AEG is a registered trademark used under license from AB Electrolux (publ)